
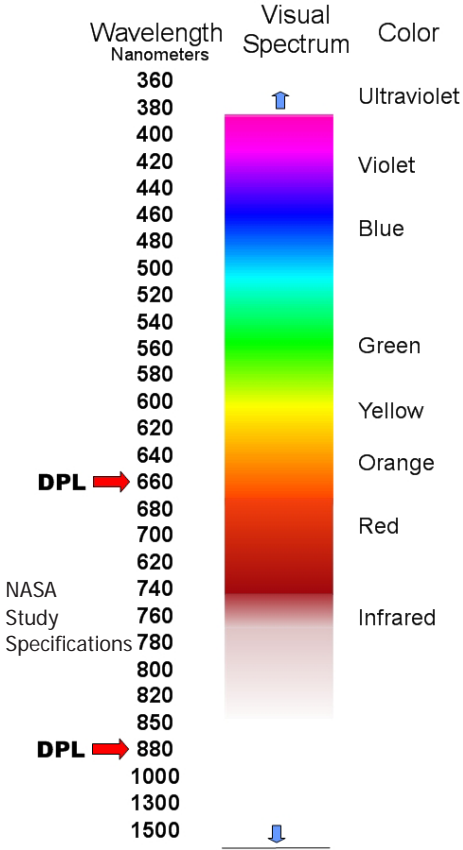


Relief from Pain & Sore Muscles

LED Technologies, LLC manufacturers the DPL™ Therapy System for the treatment of pain, relief of muscle and joint aches, sprains, back pain and the pain and stiffness associated with arthritis.

The DPL™ System uses a form of energy called "Photons" to penetrate deep into the body easing pain and repairing damaged tissues. The DPL™ Therapy System is a non-invasive, drug free therapy which can be applied to your specific and individual needs.

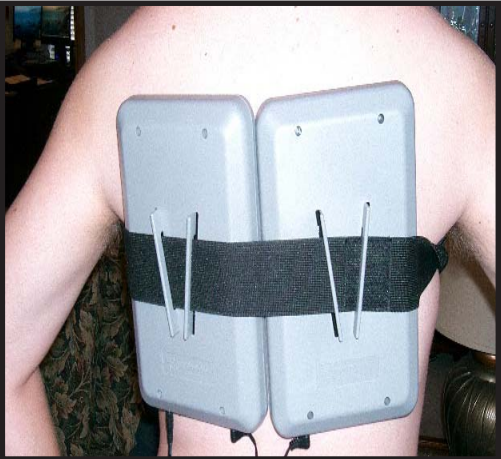

Light therapy has been shown in over 40 years of independent research worldwide to deliver powerful therapeutic benefits to living tissues and organisms. Both visible RED and INFRARED light have been shown to affect at least 24 different positive changes at a cellular level. Visible RED light at a wavelength of 660nm penetrates tissue to a depth of about 8-10 mm. It is very beneficial in treating problems close to the surface of the skin such as wounds, cuts, scars, and treating infection. INFRARED light at 880nm penetrates to a depth of about 30-40 mm which makes it more effective for bones, joints and deep muscle problems.

Benefits of DPL™ Therapy



DPL™ Therapy activates production of endorphins and blocks pain-transmitting chemicals, causing non-narcotic analgesia. Recent studies have proven the efficacy of 660nm and 880nm LEDs in the management of sports and muscle injuries, and pain.



Accelerates Healing for the Following Conditions:

- ✓ Tennis Elbow
- ✓ Golfers Elbow
- ✓ Bone Spurs
- ✓ Bone Chips
- ✓ Leg Pain
- ✓ Carpal Tunnel Syndrome
- ✓ Wrist Pain
- ✓ Shoulder Pain
- ✓ Sore Back
- ✓ Tight Muscles
- ✓ Arthritis
- ✓ Eliminate Post-Event Stress
- ✓ Increase Range of Motion
- ✓ Tendon Problems
- ✓ Deep Muscle Problems
- ✓ Swelling
- ✓ Bruises
- ✓ Ankle Problems
- ✓ Plantar Fasciitis
- ✓ Tissue Repair
- ✓ Bone Fractures
- ✓ Inflammation
- ✓ Articulations
- ✓ Nerve Injuries
- ✓ Strains & Stress
- ✓ TMJ
- ✓ Spasms & Knots
- ✓ Arthritic Pain
- ✓ Muscle Atrophies
- ✓ Cartilage Wear
- ✓ Bursitis
- ✓ Bedsores
- ✓ Neuralgia
- ✓ Speed Healing of Hematomas
- ✓ Pain Relief
- ✓ Prevent Formation of Scar Tissue
- ✓ Reduce Existing Scar Tissue
- ✓ Neck Pain & Stiffness
- ✓ Ligament Tears
- ✓ Torn Cartilage

The DPL™ Therapy System uses specific LEDs that match the NASA Study protocol. The DPL™ Therapy System is engineered to treat pain and sore muscles and accelerate healing in order to reduce or eliminate the need for potentially harmful drugs and expensive and invasive surgery.


Using the DPL™ Therapy System to Treat Pain and Muscle Soreness:

Panel Strap Assembly:


Slide Velcro strap through opening on back of panels (illustrated below). You can use the Master Panel alone if only one panel is needed, or connect both panels together with the Velcro strap and panel connector wire.



You can then strap the panel on just about any part of your body (illustrated below).

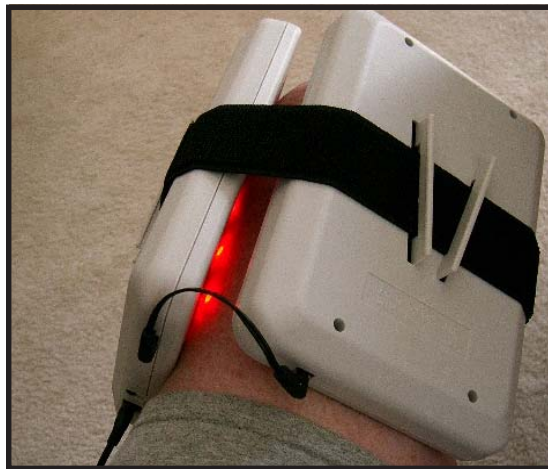

You can also rest the Panel(s) on or around the muscle or pain that is bothering you.

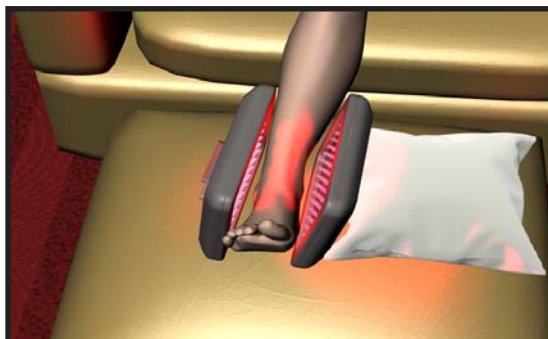

Frequently Asked Questions: Pain Relief

Question: What will I notice during and after treatment?

1. You will feel an increase in warmth in the treated areas.
2. Different sensations are possible with use of the system depending on the individual.

You may experience the relief of pain.

You may have a greater range of motion.

You may feel a calming or soothing effect.

Question: When will I feel better or see some positive results?

It is important to remember that we are all unique; no two people are the same. Factors such as age, sex, level of fitness and various physical conditions can affect how quickly your system can help. In cases where your condition has lasted for many years (chronic) it may take longer to notice a benefit because so many areas may need the therapeutic benefits of the DPL™ system.

Question: How often can I use the DPL™ System?

For pain relief maximum use of the system should be four 9 minute treatment sessions, with 12 hours separating the first two 9 minute treatments.

Question: Can the DPL™ Therapy System help my (insert your problem)?

DPL™ Therapy can be used for the treatment of chronic pain by emitting energy in the Near-IR spectrum for the temporary relief of minor muscle and joint pain; arthritis and muscle spasm; relieving stiffness; promoting relaxation of muscle tissue; and temporarily increase local blood circulation where applied.

Question: How close to the skin should I hold the DPL™ System?

For best results the DPL™ should be as close to the skin as possible, the outside plastic of the panels can even touch the skin. Center the panel(s) to the main area of pain for maximum effectiveness.

Question: I have arthritis in my hands. Will I be able to use the system?

The DPL™ System is designed with the arthritis sufferer in mind and can easily be used on the stand for your hands. The device is also lightweight and easy to hold. Treatment should focus on the hands until more mobility is obtained before moving on to other areas of the body.

Question: I have used my system for a couple of days - why am I still hurting?

Keep in mind that this system serves as therapy for minor muscle, joint and arthritic discomfort; it is not a miracle drug or pain killer. The system is not intended to replace any current medication prescribed by your physician. Always consult your physician if the pain persists or intensifies. This may be a symptom of other conditions needing immediate attention.

Question: Does it work through clothes?

Some materials can prohibit the system from delivering the treatment most effectively. If possible, use without clothes on the treatment area.

Question: Why doesn't the system get hot?

Some people report that their system gets warm or they feel warmth in their skin, which is normal. When you follow the recommended use, the LEDs should not get uncomfortable or dangerously hot.